

Adapt Lubricating Deodorant

General Information

- **Adapt** lubricating deodorant neutralizes odors, eases emptying, and helps prevent pouch static and sticking
- It is not a perfume or masking agent
- Adapt lubricating deodorant should be used in drainable and closed pouching systems only

For Drainable Pouches

1. Apply pouch.
2. Bottle: add approximately 5 mL or 1 tsp to bottom of pouch (Figure 1).
Packet: add the complete contents of the packet to bottom of pouch (Figure 2). Close the pouch in the normal manner.
3. Rub to thoroughly coat the inside of the pouch (Figure 3). Repeat Step 2 every time you empty your pouch.

For Closed Pouches

1. Add Adapt lubricating deodorant before removing the release liner on your skin barrier.
2. Bottle: add approximately 5 mL or 1 tsp through pouch opening (Figure 4).
Packet: add the complete contents of the packet through pouch opening.
3. Apply pouch.
4. Rub to thoroughly coat the inside of the pouch (Figure 3).



FREQUENTLY ASKED QUESTIONS

Does it matter if the Adapt lubricating deodorant comes in contact with the filter on the pouch?

When coating the inside of the pouch, it is best to avoid the filter.

Do I need to use the Adapt lubricating deodorant every time I empty my pouch?

Yes. For this ostomy accessory to work effectively, it must be added each time your pouch is drained.

Can I just put a few drops in the bottom of my pouch and not rub it against the sides of the pouch?

No, this is not recommended since it will not be effective as a pouch lubricant or as a pouch deodorant. Rubbing the pouch helps to coat the inside of the pouch film.

Isn't it easier to use a cooking spray or oil instead to coat the inside of my pouch?

This is not recommended because cooking oil may cause the seal of your pouch to fail. Also, cooking oil sprays do not eliminate odor.

Adapt Lubricating Deodorant

FREQUENTLY ASKED QUESTIONS

| | |
|---|--|
| <p>The 8 oz bottle is not convenient to take into a public restroom. Is there a smaller bottle available?</p> | <p>Adapt lubricating deodorant is also available in single-use packets. This is much more discrete and convenient when you are away from home.</p> |
| <p>It doesn't have a scent. How do I know it is working?</p> | <p>Adapt lubricating deodorant is not a perfume or a fragrance used to mask or cover-up odors. It does not have any odor by itself, but it will neutralize odors in the pouch.</p> |
| <p>Can the Adapt lubricating deodorant come in contact with the stoma?</p> | <p>Yes, it has been tested and it will not harm your stoma.</p> |
| <p>What if Adapt lubricating deodorant comes in contact with my skin?</p> | <p>This product is not a skin irritant. It is water-soluble, so it will easily be removed from the skin with a moist cloth. Be sure to wash it off your skin before applying your pouch, so it does not interfere with adhesion.</p> |



Adapt lubricating deodorant

Stock No 78500
(8 oz bottle)

Stock No 78501
(box of 50, 8 mL packets)

Routine follow-up with your healthcare professional is recommended.

Indications and Intended Use: This product is an accessory to an ostomy pouching system (e.g., colostomy, ileostomy) intended to help minimise (minimize) odour (odor) and to facilitate pouch emptying.

Precautions: Do not use with any other pouch additive. Do not use on wounds or take internally. Should skin irritation occur, discontinue use and wash the skin carefully. Keep away from infants and children.

For Bottle (Single patient - multiple use): Sharing the product with multiple users creates risk of infection. May use until bottle is empty.

For Packet (Warning): To help reduce the potential for infection and/or other complications, do not reuse.

Notes: Avoid contact with the adhesive of the skin barrier. Avoid contact with the filter. Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.hollister.com/authority or contact EC Rep or local distributor.

Instructions for Use: To Apply -

Step 1 Bottle: Add approximately 5 mL or 1 tsp.

Packet: Add the complete contents of the packet. Close the pouch in the normal manner (if applicable).

Step 2 Rub to thoroughly coat the inside of the pouch. Note: for drainable pouches repeat Steps 1 and 2 after every emptying.

To Dispose: Please follow all local disposal regulations.

CE Not all products are CE marked.

The Hollister logo, Adapt and "Healthy skin. Positive outcomes." are trademarks of Hollister Incorporated.
© 2020 Hollister Incorporated.

Hollister Incorporated
2000 Hollister Drive
Libertyville, Illinois 60048
1.800.323.4060

www.hollister.com