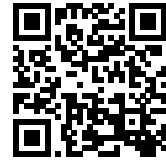
Guide for Use

Onli™
Intermittent Catheter

Continence Care

Onli™
Intermittent Catheter


For more information visit
[Hollister.com/Onli](https://www.hollister.com/Onli)


1 Wash hands.

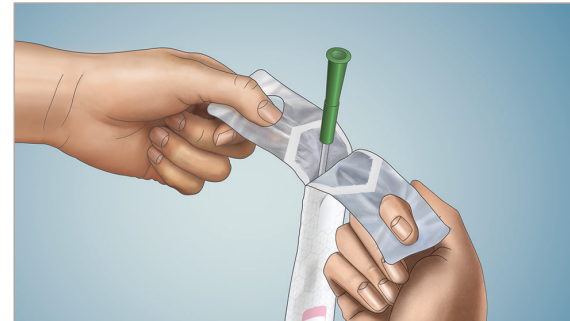
2 Spread the labia and cleanse around opening of urethra with mild, unscented soap or a non-alcoholic wet wipe as instructed by your healthcare practitioner. Choose a position to catheterize that is most comfortable for you (standing, sitting on the toilet, in wheelchair or lying in bed).

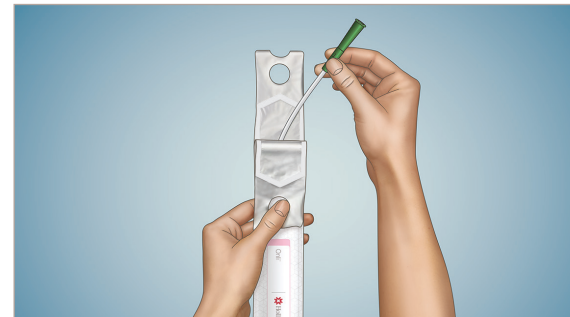
3 Using finger holes peel open the package until the funnel is visible.


4 Hold funnel to remove catheter.

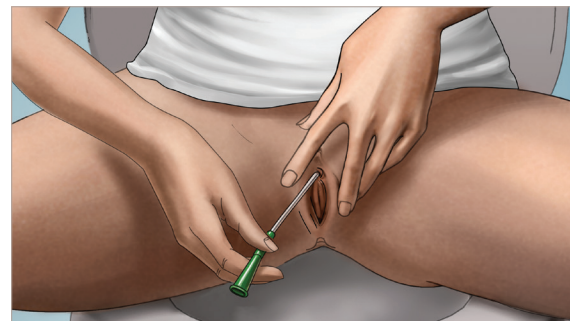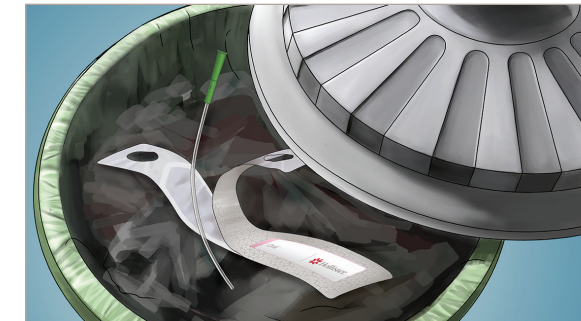5 Spread labia with two fingers while inserting catheter into the urethra. Hold funnel, without touching the catheter and insert catheter until urine starts to flow.

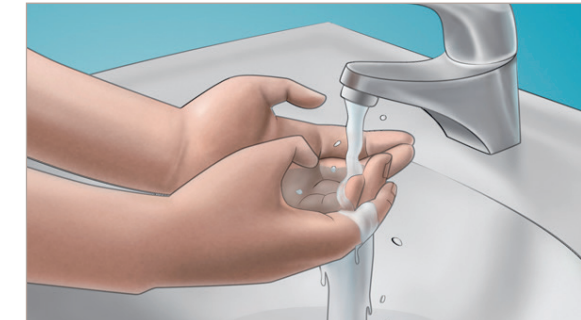6 Once urine flow has stopped slowly withdraw the catheter. If urine flows again, pause until urine stops. After you're sure the bladder is empty, remove the catheter completely. Catheter may be disposed of in a waste bin. **DO NOT FLUSH DOWN TOILET.** Please follow all local disposal regulations.

7 Wash hands.

Indications and Intended Use: This intermittent catheter is a flexible tubular device that is inserted through the urethra by male, female and pediatric patients who need to drain urine from the bladder.

WARNINGS

Do not use if package is damaged or opened. To help reduce the potential for infection and/or other complications, do not reuse. If discomfort or any sign of trauma occurs, discontinue use immediately and consult your healthcare practitioner.

Precautions: Please consult your healthcare practitioner before using this product if any of the following conditions are present:

- Severed urethra
- Unexplained urethral bleeding
- Pronounced stricture
- False passage
- Urethritis – inflammation of the urethra

Self-catheterization should follow the plan of care and advice given by your healthcare practitioner and be carried out only in accordance with the instructions for use provided. Because catheterization frequency varies by person, the recommended frequency of your catheterization should be provided by your healthcare practitioner. For any other questions about your catheterization, please contact your healthcare practitioner.

Notes: Store boxes in a flat position and at normal room temperature. Contact a healthcare practitioner for help with continence care products. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.hollister.com/authority or contact EC Rep or local distributor.

USA: Rx Only.

Caution: Federal (USA) Law restricts this device to sale by or on the order of a physician or other healthcare practitioner licensed under state law to order this product.

Hollister, the Hollister logo and Onli are trademarks of Hollister Incorporated.
© 2022 Hollister Incorporated. 923388-0822 US-01767

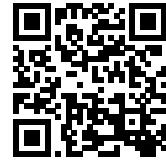
Hollister Incorporated
2000 Hollister Drive
Libertyville, Illinois 60048
1.888.808.7456

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